**Sweet
compress**

☐ **PAD MANO**
HAND PAD
PAD MAIN
Cod. TNCZ0

☐ **IMBOTTITURA CAVO GOMITO**
ELBOW PIT PADDING
REMBOURRAGE CREUX COUDE
Cod. TNPR1

☐ **IMBOTTITURA CAVO POPLITEO**
POPLITEAL FOSSA PADDING
REMBOURRASGE CREUX POPLITÉ
Cod. TNPR0

CIZETA

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INTENDED USE AND TARGET PATIENTS: accessories for SWEETCompress®, self-bandage recommended for the therapy of oedema aimed to patients suffering from primary and secondary lymphoedema, oedema.

HAND PAD: pad in open-cell polyurethane foam, externally covered with cotton with two finger openings. It can be positioned on the back of the hand, to increase exerted pressure. Measures: 8.5 x 24cm

ELBOW PIT PADDING: padding in open-cell polyurethane foam externally covered with cotton. It can be positioned at the elbow for increased comfort in the joint area. Measures: 10 x 13cm

POPLITEAL FOSSA PADDING: padding in open-cell polyurethane foam externally covered with cotton. It can be positioned at the back of the knee for increased comfort in the joint area. Measures: 13 x 17cm

Composition: outer layers: cotton 93%, elastane 7%; inner layer: polyurethane foam PU 100%.

Contraindications to use: immobility/severe hypomobility; skin infections in the acute phase; exudative dermatitis; acute dermo-hypo dermatitis; diseases of the lower limbs with no venous-lymphatic origin; acute or undiagnosed oedemas with no venous origin (dysmetabolic, diabetic, renal, cardiac, endocrine...); known contact allergy to medical compression

stockings and their components; congenital/acquired anatomical anomalies (except for the possible use of custom-made braces). Before wearing it, dress any open wounds in the area to be treated. Cizeta Medicali is not responsible for the failure to comply with the above contraindications.

Washing instructions:

- Machine washing with temperature not exceeding 60°C
- Use detergent for delicate clothes
- Do not bleach
- Do not use fabric softeners
- Do not dry in automatic dryers and/or in the sun
- Pat dry with a towel to remove excess water
- Hang horizontally
- Keep away from heat sources
- Do not iron, not even steam iron
- Do not dry clean and do not use solvent-based stain removers

Product storage: keep away from direct sunlight.

Management of inappropriate devices: if the device is damaged, notify the manufacturer immediately.

Side effects and serious accidents: no side effects are known to date. Should negative episodes (e.g., skin irritation) arise during use, see your doctor. In case of a serious accident occurring in connection with the device, notify immediately the manufacturer and the competent health professional.

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