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EN - COMPRESSION ML20LINES
INTENDED USE AND TARGET PATIENTS: indicated as support bandages in case of bruises, sprains, dislocations and in case of tendinopathies, for immobilisation of joints, protection against hematomas and in case of muscle injuries. Indicated for bandages in case of lymphatic and lipo-lymphatic diseases and Chronic Venous Insufficiency (CVI). **COMPOSITION:** 80% cotton, 20% polyamide. Latex free. **CONTRAINDICATIONS:** this medical device should not be applied directly on injured tissues. **INSTRUCTIONS FOR USE:** the dressing should be performed by competent staff. **MANAGEMENT OF DAMAGED DEVICES:** if the device is damaged, notify the manufacturer immediately. **SIDE EFFECTS AND SERIOUS ACCIDENTS:** there are no known allergies, except for the specific ones to the individual textiles components. The dressing should not be performed on pathological conditions advising against compressed bandages. In case of a serious accident occurring in connection with the device, notify the manufacturer and the competent doctor immediately. **STORAGE:** keep away from direct light and sources of heat; store it preferably in a cool and dry place at a temperature below 25°C.

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