ANATO C1

INDICATIES:

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ANATO C1

INDICATIONS:

- Torticollis
- Cervical arthrosis
- Gradual weaning off a more rigid device

FEATURES:

Flexible, analgesic support of the cervical spine.
Comfort and efficacy enhanced by an indentation and an integrated chin-rest.
Bottom edge stitching for a better fit.

CONTRAINDICATIONS:

- Severe traumatic injury (Fracture, Sprain)
- Major respiratory failure
- Must not come into direct contact with broken skin

FITTING THE DEVICE:

- 1 Position the brace around the neck.
 - 2 Secure the brace by the self-adhesive closure.
 - 3 Make sure to centre the closure at the nape of the neck.
- N.B.: The choice of model - 8 cm H or 10 cm H - results in the correct head position ("Horizontal" vision).

PRECAUTIONS:

Follow the advice of the health professional who prescribed or supplied the product. If there is doubt regarding use, or discomfort, consult your health professional.

The orthotic device is for personal use.

Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

To throw away the product, respect strictly the legal regulations of your region.

MAINTENANCE:

Hand wash in 40° water with mild, neutral soap.
Rinse thoroughly without spinning, and lay flat to dry, away from excessive heat.
Do not iron.

PRESENTATION:

Navy blue
Height: 8 or 10 cm
3 sizes: (Neck circumference)

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