ANATO C2

INDICATIES:

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ANATO C2

INDICATIONS:

- Stubborn torticollis
- Cervical arthrosis
- Mild cervical sprain
- Cervicobrachial syndrome

FEATURES:

Analgesic support conforming to the morphology of the cervical spine (30 kg/m3 inner foam)
Comfort and efficacy enhanced by an indentation and an integrated chin-rest.
Bottom edge stitching for a better fit.

CONTRAINDICATIONS:

- Severe traumatic injury (Fracture, Moderate Sprain)
- Major respiratory failure
- Must not come into direct contact with broken skin

FITTING THE DEVICE:

- 1 Position the brace around the neck.
 - 2 Secure the brace by the self-adhesive closure.
 - 3 Make sure to centre the self-adhesive closure at the nape of the neck.
- N.B.: The choice of model - 8 cm H or 10 cm H - results in the correct head position ("Horizontal" vision).

PRECAUTIONS:

Follow the advice of the health professional who prescribed or supplied the product. If there is doubt regarding use, or discomfort, consult your health professional.
The orthotic device is for personal use.
Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.
To throw away the product, respect strictly the legal regulations of your region.

MAINTENANCE:

Hand wash in 40° water with mild, neutral soap.
Rinse thoroughly without spinning, and lay flat to dry, away from excessive heat.
Do not iron.

PRESENTATION:

Navy blue
Height: 8 or 10 cm
3 sizes: (Neck circumference)

- Size 1: 28 - 33 cm
- Size 2: 34 - 39 cm
- Size 3: 40 - 45 cm

MATERIALS:

Polyurethane 52% - Cotton 20% - Polypropylene 20% - Polyamide 6% - Polyethylene 1% - Acrylic 1%

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