ANATO C3

INDICATIES

- Halstrauma
- Matige tot ernstige verstuiking
- Cervicobrachiale neuralgie na trauma

KENMERKEN

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ANATO C3

INDICATIONS

- Neck trauma
- Moderate to severe sprains
- Traumatic cervicobrachial syndrome

FEATURES

Brace comprising 2 polyethylene micro-mesh plates for reinforcement and ventilation.
Height-adjustable neck brace, with chin-rest.
Radio-transparent brace.
Leatherette-covered contours and chin-rest, with inner polyurethane foam lining for optimal comfort.
Self-adhesive closure.

CONTRAINDICATIONS

- Severe neck trauma requiring more rigid immobilisation with a C4 neck brace
- Must not come into direct contact with broken skin

FITTING THE DEVICE

- 1** Adjust the height of the 2 polyethylene plates so that the chin is correctly positioned in the chin-rest.
- 2** Place the brace around the neck.
- 3** Secure the brace by the self-adhesive closure.
- 4** Make sure to centre the self-adhesive closure at the nape of the neck.

PRECAUTIONS

Follow the advice of the health professional who prescribed or supplied the product. If there is doubt regarding use, or discomfort, consult your health professional.
The orthotic device is for personal use.
Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.
To throw away the product, respect strictly the legal regulations of your region.

MAINTENANCE

Hand wash in cold water with mild, neutral soap.
Rinse thoroughly without spinning, and lay flat to dry, away from excessive heat.
Do not iron.
Do not use bleach.

PRESENTATION

White
4 sizes: (Neck circumference and Height)
- Size 1: 25-30 cm and 6-9 cm - Size 3: 35-40 cm and 9-12 cm
- Size 2: 30-35 cm and 7.5-10.5 cm - Size 4: 40-45 cm and 10-13 cm

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