ANATO C4

INDICATIES:

- Ernstig halstrauma
- Ernstige verstuiking

KENMERKEN:

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- Severe neck trauma
- Severe sprain

FEATURES:

Brace comprising 2 anatomically shaped sections in EVA foam, combining shock-resistance and flexibility.
Chin-rest. Occipital, dorsal and sternum support.
Radio-transparent brace.
Brace adjustable using a heat gun.
Chin-rest for restricting head movement.
Wide opening for tracheostomy
Velcro closure.

CONTRAINDICATIONS:

- Allergic reaction to EVA
- Must not come into direct contact with broken skin

FITTING THE DEVICE:

- Put the back section in position.
- Position the front section over the back section.
- Adjust the front and back sections in order to keep the neck correctly aligned.
- Secure the brace by means of the self-adhesive closures on both sides of the product, taking care to adjust the tightness.

N.B.: A hot air gun may be used to adjust the foam to optimally fit the patient.

PRECAUTIONS:

Follow the advice of the health professional who prescribed or supplied the product. If there is doubt regarding use, or discomfort, consult your health professional.

The orthotic device is for personal use.

Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

MAINTENANCE:

Hand wash in cold water with mild, neutral soap.
Rinse thoroughly without spinning, and lay flat to dry, away from excessive heat.
Do not iron.
Do not use bleach.

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