ATL SEAMLESS POUCE

INDICATIES:

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ATL SEAMLESS POUCE

INDICATIONS:

- Post-traumatic immobilisation of the metacarpophalangeal joint of the first finger
- Post-traumatic immobilisation of the trapeziometacarpal joint of the thumb
- Root arthrosis of the thumb in the hyperalgesic phase

FEATURES:

Immobilisation and resting of the thumb column.
Breathable fabric with anatomical cut-out.
Detachable radiolucent aluminium stays.
Ambidextrous orthotic device.

CONTRAINDICATIONS:

- Some deformities cannot be accommodated by standard splints
- Must not come into direct contact with broken skin

FITTING THE DEVICE:

1 Pull on the splint.

Centre and fasten without over-tightening, by means of the self-adhesive straps.

2 Shape the frame to fit the morphology of the patient.

The stays will be correctly positioned by your doctor or the person applying the device.

PRECAUTIONS:

Read the instructions carefully before using the orthotic device.

Only a doctor is authorised to prescribe treatment and determine its duration.

The orthotic device is for personal use. After being fitted on the patient, it should not be modified, except by qualified specialists.

The general instructions and special directions prescribed by the doctor must be followed at all times.

Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

To throw away the product, respect strictly the legal regulations of your region.

MAINTENANCE:

It is advisable to clean the Velcro® straps and keep them free of fibres, lint, etc. so as to preserve their adhesive properties.

Hand wash. Do not use bleach. Do not iron. Do not dry clean. Do not tumble dry.

PRESENTATION:

2 sizes: (Circumference of the wrist and thumb)

- Size 1: 14 - 18 cm and 5 - 7 cm
- Size 2: 18 - 22 cm and 6 - 8 cm

MATERIALS:

Polyurethane 44% - Polyamide 30% - Cotton 23% - Elastane 3%

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