CIROM

INDICATIES:

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CIROM

INDICATIONS:

- Sprain of the PCL, ACL, ILL, ELL
- Post-operative following a knee ligamentoplasty
- Post-repair of the meniscus
- Knee surgery
- Rehabilitation phase

FEATURES:

Knee brace with flexion/extension adjustable and height adjustable joint.

4 padded cushions.

Straps can be cut to the desired length thanks to detachable velcro.

CONTRAINDICATIONS:

- Knee deformities do not allow the fitting of this brace
- Circumferences greater than 48cm at the calf and greater than 71cm at the thigh do not allow this brace to be worn
- Must not be in direct contact with a skin lesion

FITTING THE DEVICE:

- 1 Open the fasteners and undo the straps.
- 2 Position the brace, taking care to centre the joints at knee level.

Note: The "TOP" marking visible on the plastic indicates the correct direction for positioning the product (from the thigh towards the calf). Ensure that the calf support cushions (the smaller cushions) are oriented toward the foot.

- 3 Adjust the height on the lateral supports. Ensure that the lateral supports are of the same height to avoid injury to the patient.

Note: To adjust the height, press the release tab, adjust the plastic bar to the desired height, then release the release tab. The height of the brace can be checked using the length indicators on the lateral supports.

- 4 Fasten the straps and carry out a final tightening around the knee circumference. Put the other straps in place and secure everything.

- 5 Close the fasteners.

Note: The straps can be cut to the desired length.

- 6 To adjust the extension, pull the tab outwards and move it to the desired setting between 0° and 90°, then release the tab to lock the position.

- 7 To adjust the flexion, pull the tab outwards and move it to the desired setting between 0° and 120°, then release the tab to lock the position.

- 8 Joints can be locked at 0°, 15° or 30° by pressing the quick release button.

PRECAUTIONS:

Follow the advice of the healthcare professional who prescribed or issued the product. If in doubt about use, or in case of discomfort, consult your healthcare professional.

This orthosis is for personal use.

Contact the manufacturer and the competent authority in case of a serious incident arising from use of this device.

To dispose of the product, strictly comply with the legal regulations of your region.

MAINTENANCE:

Detach the textile part from the frame.

Wash by hand in 40° water with neutral, non-aggressive soap.

Rinse thoroughly without wringing and leave to dry flat, away from a source of excessive heat.

PRESENTATION:

- Black
- Single size
- Length adjustable from 48 to 70cm
- Min. – max. calf circumference: 25 - 50cm
- Min. – max. thigh circumference: 30 - 70cm

MATERIALS:

Polyamide plastic 32% - Aluminium 30% - EVA foam 20% - Polyamide fabric 12% - Polyamide velcro 6%

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