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INDICATIES:

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INDICATIONS:

ENG

- Scheuermann's disease
- Kyphotic posture
- Stable vertebral fractures of the lower dorsal spine and the lumbar spine.
- Vertebral pain caused by secondary metastasis
- Medium-term immobilisation following intervertebral disc surgery
- Post-surgical treatment following spinal surgery, by decompression with or without internal fixation

FEATURES:
Lightweight aluminium structure.
Removable sternum plate.
Dorsolumbar tension plate that can be adjusted using the strap and closure system.
3 support points: pectoral, suprapubic and dorsolumbar.
Height adjustable by laterally positioned control fittings.
Padded dorsolumbar plates for increased comfort.
Swivelling pelvic strap that adapts to the patient's various positions.

CONTRAINDICATIONS:
- Irreducible kyphosis
- Must not come into direct contact with broken skin

FITTING THE DEVICE:
1 Alter the metal frames, adjusting the width and height by the screws.
2 Detach the strap from the pressure buckle, by pressing the switches on both sides.
3 Put the spinal brace in place, taking care to correctly arrange it against the pelvis.
4 5 Insert the back strap into the pressure buckle and adjust the tightness until compression is adequate.
The patient should wear this device directly on a garment.

PRECAUTIONS:
Follow the advice of the health professional who prescribed or supplied the product. If there is doubt regarding use, or discomfort, consult your health professional.
The orthotic device is for personal use.
Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

MAINTENANCE:
Hand wash in cold water with mild, neutral soap.
Rinse thoroughly without spinning, and lay flat to dry, away from excessive heat.
Do not iron.

Do not use bleach.
PRESENTATION:
White
4 Sizes: Hip circumference and Height (from the V of the sternum to the pelvis)
- Size 1: 60-75 cm and 39-46 cm - Size 3: 90-105 cm and 45-52 cm
- Size 2: 75-90 cm and 42-49 cm - Size 4: 105-115 cm and 49-56 cm

MATERIALS:
Aluminium 60% - Polyamide 15% - Polyvinyl chloride 10% - Polyurethane 10% - Polypropylene 3%
- Polyoxymethylene 2%

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