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NL

INDICATIES:

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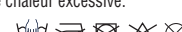
ENG

INDICATIONS:

- Post-traumatic or post-surgical shoulder support
- Rotator cuff surgery
- Dislocation of the joint capsule
- Following the Bankart surgical procedure
- Luxation and subluxation of the shoulder joint
- Soft tissue distension and repair

FEATURES:

Enables the arm to be placed in several positions as required post-surgically.

Restriction of internal rotation: elbow movement is possible while wearing the device.

Stabilisation and comfort owing to the cushion's adjustable anatomical shape.

3D mesh outer covering and sponge fabric on the skin contact side, eliminating moisture for greater comfort and to prolong wear.

Very easily fitted.

Support tab for thumb.

Strap height adjustable by a buckle for an optimal fit.

Model adaptable to either side.

CONTRAINDICATIONS:

- Allergic reaction to textile materials
- Not suitable for desired angle settings wider than 20°
- Must not come into direct contact with broken skin

FITTING THE DEVICE:

1 Arrange the forearm inside the sleeve.

If desired, place the thumb in the support tab.

2 Adjust and tighten the shoulder strap for optimal restraint.

If desired, place the thumb in the support tab.

3 Place the abduction cushion in the desired position.

4 Attach the sleeve to the abduction cushion.

4 Adjust and tighten the girdle strap for optimal support.

The patient should wear this device directly on a garment.

PRECAUTIONS:

Follow the advice of the health professional who prescribed or supplied the product. If there is doubt regarding use, or discomfort, consult your health professional.

The orthotic device is for personal use.

Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

To throw away the product, respect strictly the legal regulations of your region.

MAINTENANCE:

Hand wash in 40° water with mild, neutral soap.

Rinse thoroughly without spinning and lay flat to dry, away from excessive heat.

Do not iron.

PRESENTATION:

Grey

One Size Only

MATERIALS:

Polyurethane 70% - Polyester 20% - Polyamide 10%

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