GL 120 STANDARD

INDICATIES:

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GL 120 STANDARD

INDICATIONS:

- Moderate to chronic knee instability
- MCL/LCL sprain
- After-effects of trauma
- Post-surgical protection
- Resumption of sporting activities
- Patella syndrome

FEATURES:

Knee brace with polycentric hinge, for knee stabilisation during exertion.

Anatomical cut-out for optimal comfort.

Patella recess and shock-absorber promoting stability of the kneecap.

CONTRAINDICATIONS:

- Must not come into direct contact with broken skin

FITTING THE DEVICE:

- 1 Loosen the straps at the thigh and calf.
- 2 Pull on the hinged knee brace, with the label on the inside.
- 3 Position the hinges on either side of the knee, with the patella ring arranged over the kneecap.
- 4 Fasten the straps around the thigh and calf.

The patient should wear this device directly on a garment.

PRECAUTIONS:

Follow the advice of the health professional who prescribed or supplied the product. If there is doubt regarding use, or discomfort, consult your health professional.

The orthotic device is for personal use.

Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

To throw away the product, respect strictly the legal regulations of your region.

MAINTENANCE:

Hand wash in 40° water with mild, neutral soap.

Rinse thoroughly without spinning, and lay flat to dry, away from excessive heat.

Do not iron.

PRESENTATION:

6 sizes: (Circumference of thigh and half-bent knee)

- | | |
|-------------------------------------|-------------------------------------|
| - Size 1: 33 – 38 cm and 30 – 32 cm | - Size 4: 52 – 57 cm and 39 – 41 cm |
| - Size 2: 38 – 45 cm and 33 – 35 cm | - Size 5: 57 – 63 cm and 42 – 44 cm |
| - Size 3: 45 – 52 cm and 36 – 38 cm | - Size 6: 63 – 70 cm and 45 – 48 cm |

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