GL STAB

INDICATIES:

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FEATURES:

Created on a flat loom and anatomically pre-shaped, the GL Stab knee brace provides optimal comfort by reducing the risk of a tourniquet effect.
Class 3 compression.
Knitted fabric with distinct compression zones.
Limited compression on the patella.
Patella recess for effectively maintaining the kneecap's position.
Soft knitted fabric at the popliteal fossa to eliminate stricture and the tourniquet effect.

CONTRAINDICATIONS:

- Some deformities cannot be accommodated by standard splints
- A.O.L.L (Arteriosclerosis Obliterans of the Lower Limbs)
- Must not come into direct contact with broken skin

FITTING THE DEVICE:

Pull on the knee brace until the patella recess is positioned on the kneecap.
Adjust the knee brace, as needed, for optimal comfort.

PRECAUTIONS:

Follow the advice of the health professional who prescribed or supplied the product. If there is doubt regarding use, or discomfort, consult your health professional.
The orthotic device is for personal use.
Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.
To throw away the product, respect strictly the legal regulations of your region.

MAINTENANCE:

Hand wash in 40° water with mild, neutral soap.
Rinse thoroughly without spinning, and lay flat to dry, away from excessive heat.
Do not iron.

PRESENTATION:

- 7 sizes: (Circumference of thigh and half-bent knee)
- Size 1: 38 – 40 cm and 30 – 32 cm
 - Size 2: 41 – 43 cm and 33 – 35 cm
 - Size 3: 44 – 47 cm and 36 – 38 cm
 - Size 4: 48 – 52 cm and 39 – 41 cm
 - Size 5: 53 – 57 cm and 42 – 44 cm
 - Size 6: 58 – 62 cm and 45 – 49 cm
 - Size 7: 63 – 67 cm and 50 – 54 cm

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