GL STRAPP

NL

INDICATIES:

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ENG

INDICATIONS:

- Moderate chronic ligament laxity
- Residual post-traumatic oedema
- After-effects of trauma
- Moderate osteoarthritis
- Moderate patella syndrome
- Resumption of sporting activities
- Mild sprain
- Protection from movement exposing to risk of trauma

FEATURES:

Created on a flat loom and anatomically pre-shaped, the GL Strapp knee brace provides optimal comfort by reducing the risk of a tourniquet effect.

Class 3 compression.

Knitted fabric with distinct compression zones.

Limited compression on the patella.

Innovatively shaped patella ring.

Patella ring stimulating active realignment of the kneecap.

Patella recess for effectively maintaining the kneecap's position.

Soft knitted fabric at the popliteal fossa to eliminate stricture and the tourniquet effect.

Strapping comprising 3 straps.

Increased proprioception and patella ring efficacy thanks to the strapping.

CONTRAINDICATIONS:

- Some deformities cannot be accommodated by standard splints
- A.O.L.L. (Arteriosclerosis Obliterans of the Lower Limbs)
- Must not come into direct contact with broken skin

FITTING THE DEVICE:

1 Pull on the knee brace until the kneecap is positioned in the centre of the patella ring.

Adjust the knee brace, as needed, for optimal comfort.

2 and **3** Pull on the lower straps and attach them on either side of the product to form a "V".

4 Pull on the upper strap and attach it in a circular fashion.

5 Ensure that there are no creases in the straps, for greater comfort.

PRECAUTIONS:

Follow the advice of the health professional who prescribed or supplied the product. If there is doubt regarding use, or discomfort, consult your health professional.

The orthotic device is for personal use.

Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

To throw away the product, respect strictly the legal regulations of your region.

MAINTENANCE:

Hand wash in 40° water with mild, neutral soap.

Rinse thoroughly without spinning, and lay flat to dry, away from excessive heat.

Do not iron.

PRESENTATION:

7 sizes: (Circumference of thigh and half-bent knee)

- | | |
|-------------------------------------|-------------------------------------|
| - Size 1: 38 – 40 cm and 30 – 32 cm | - Size 5: 53 – 57 cm and 42 – 44 cm |
| - Size 2: 41 – 43 cm and 33 – 35 cm | - Size 6: 58 – 62 cm and 45 – 49 cm |
| - Size 3: 44 – 47 cm and 36 – 38 cm | - Size 7: 63 – 67 cm and 50 – 54 cm |
| - Size 4: 48 – 52 cm and 39 – 41 cm | |

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