HYPOGRIPP

INDICATIES:

- Lichte of gemiddelde abdominale ptosis
- Fixatie na operatie
- Matige obesitas of buikwandhernia

KENMERKEN:

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HYPOGRIPP

INDICATIONS:

- Mild or moderate abdominal ptosis
- Post-surgical retention
- Moderate obesity or eventration

FEATURES:

Dynamic abdominal support assured by a combination of strong fabric and a detachable hypogastric tab.
Optimal comfort due to a dual front adjustment feature.
Microfibre material to eliminate sweat by capillary action.

CONTRAINDICATIONS:

- Marked obesity or thinness
- Major respiratory failure
- Must not come into direct contact with broken skin

FITTING THE DEVICE:

1 Centre the belt on the abdomen, with the three fanned-out springs pointing downwards.

2 Adjust the tightness by means of the two middle straps.

If necessary, attach the hypogastric tab in the appropriate place (consult your doctor or the person applying the device)

For optimal comfort, this belt is to be worn ideally over a cotton undergarment.

The patient should wear this device directly on a garment.

PRECAUTIONS:

Follow the advice of the health professional who prescribed or supplied the product. If there is doubt regarding use, or discomfort, consult your health professional.

The orthotic device is for personal use.

Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

To throw away the product, respect strictly the legal regulations of your region.

MAINTENANCE:

Hand wash in 40° water with mild, neutral soap.

Rinse thoroughly without spinning, and lay flat to dry, away from excessive heat.

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