SLEUTELBEENBRACE

INDICATIES:

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CLAVICLE SUPPORT BAND

INDICATIONS:

- Collar bone fracture or trauma with or without displacement
- Dislocation of the acromioclavicular joint

FEATURES:

- Acromioclavicular strap with a self-adhesive back fastening system.
- Adjustable traction support.
- Optimal comfort provided by the fleece-lined shoulder pads and padded shoulder strap.

CONTRAINDICATIONS:

- Must not come into direct contact with broken skin

FITTING THE DEVICE:

- 1 Place the 2 acromioclavicular straps over the shoulders, taking care to keep the armpits protected.
 - 2 Make sure to align the back fastening system between the shoulder blades.
 - 3 Put the side straps through the buckles of the back fastening system.
 - 3 Secure the straps by means of the self-adhesive closures.
- The patient should wear this device directly on a garment.

PRECAUTIONS:

- Follow the advice of the health professional who prescribed or supplied the product. If there is doubt regarding use, or discomfort, consult your health professional.
- The orthotic device is for personal use.
- Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.
- To throw away the product, respect strictly the legal regulations of your region.

MAINTENANCE:

- Hand wash in 40° water with mild, neutral soap.
- Rinse thoroughly without spinning and lay flat to dry, away from excessive heat.
- Do not iron.

PRESENTATION:

White

- 4 Sizes: (Shoulder width)
- Size 1: 23 – 28 cm
 - Size 2: 28 – 36 cm
 - Size 3: 36 – 45 cm
 - Size 4: 45 – 54 cm

MATERIALS:

- Polyurethane 24% - Polyester 24% - Polypropylene 24% - Polyoxymethylene 14%
- Polyamide 5% - PVC 4% - Steel 4% - cotton 1%

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