## LEASY POUCE

## INDICATIES:

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# LEASY POUCE

## INDICATIONS:

- Wrist trauma
- Wrist sprains
- Post-cast follow-up
- Wrist tendinopathy
- Carpal tunnel syndrome
- Wrist arthritis
- Rheumatoid arthritis

## FEATURES:

Immobilisation of the wrist in a resting position.  
Easy to use and apply thanks to its lacing system.  
Breathable fabric with anatomical shape.  
Removable, adaptable and radiolucent aluminium ribs.  
Right and left models.

## CONTRAINDICATIONS:

- Some deformities do not allow the use of a standard splint
- Must not come into direct contact with a skin lesion

## FITTING THE DEVICE:

**1** Put on the splint, taking care to apply the palmar rib to the inside of the hand.

**2A 2B** Centre the splint and gently tighten using the self-gripping straps.

The correct positioning of the ribs will be done by your doctor or your applicator.

## PRECAUTIONS:

Follow the advice of the healthcare professional who prescribed or dispensed the product.

If in doubt about use or discomfort, consult your healthcare professional.

The splint is for personal use only.

Contact the manufacturer and the competent authority in the event of a serious incident arising from the use of this device.

To throw away the product, respect strictly the legal regulations of your region.

## MAINTENANCE:

Hand wash in 40° water with a neutral, non-harsh soap.

Rinse thoroughly without wringing out, and leave to dry flat, away from excessive heat.

Do not iron.

## PRESENTATION:

3 sizes: (Wrist circumference)

- Size 0: 10 - 13 cm
- Size 1: 13 - 18 cm
- Size 2: 18 - 23 cm

## MATERIALS:

Polyester 43% - Polyamide 29% - Polyurethane 28%

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