PERFO UNI 3

NL

INDICATIES:

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PERFO UNI 3

INDICATIONS:

- Knee trauma
- Sprain
- After-effects of fractures
- Post-surgical support
- Resting (arthritis, polyarthritis)

FEATURES:

Ensures immobilisation and resting of the knee by a combination of anatomically cut self-adhesive fabric coupled with integrated detachable splints.

Optimal comfort provided by a perforated cotton terry cloth lining.

CONTRAINDICATIONS:

- Some deformities cannot be accommodated by standard splints.
- Significant varicose veins and/or ulcers
- Must not come into direct contact with broken skin

FITTING THE DEVICE:

1 2 Put the splint in place on the extended leg, with the widest section at the thigh, and the recess centred over the knee.

3 Fasten without over-tightening, by means of the self-adhesive straps.

Your doctor or the person applying the device can adjust the width (self-adhesive bands).

The patient should wear this device directly on a garment.

PRECAUTIONS:

Follow the advice of the health professional who prescribed or supplied the product. If there is doubt regarding use, or discomfort, consult your health professional.

The orthotic device is for personal use.

Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

To throw away the product, respect strictly the legal regulations of your region.

MAINTENANCE:

Hand wash in 40° water with mild, neutral soap.

Rinse thoroughly without spinning and lay flat to dry, away from excessive heat.

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