PERFO BRAS

INDICATIES:

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- Upper limb trauma
- Paralysis
- Hemiplegia
- Post-surgical immobilisation

FEATURES:

Ensures immobilisation and resting of the upper limb by a combination of anatomically cut self-adhesive fabric coupled with a stabilising band. Optimal comfort provided by a perforated cotton terry cloth lining.

CONTRAINDICATIONS:

- Must not come into direct contact with broken skin

FITTING THE DEVICE:

- 1 Put the forearm through the sleeve.
- 2 Overlap the shoulder straps at the back and stick them onto the sleeve.
- 3 Attach and wrap the stabilising band around the body.

Your doctor or the person applying the device can make the proper adjustments.
The patient should wear this device directly on a garment.

PRECAUTIONS:

Follow the advice of the health professional who prescribed or supplied the product. If there is doubt regarding use, or discomfort, consult your health professional.
The orthotic device is for personal use.
Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.
To throw away the product, respect strictly the legal regulations of your region.

MAINTENANCE:

Hand wash in 40° water with mild, neutral soap.
Rinse thoroughly without spinning, and lay flat to dry, away from excessive heat.
Do not iron.

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