IMMOBILISATIEVEST

INDICATIES:

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SAMENSTELLING:

Polyester 77% - Polyurethaan 12% - Katoen 6% - Polyamide 5%

CIZETA MEDICALI France SAS
109, rue de la Brasserie - 18200 SAINT AMAND MONTROND
Tel. 02 48 96 87 93 - Fax. 02 48 96 80 91
www.cizetamedicali.fr

Capital social 84 000 €
SIRET 398 197 863 00069 APE 4646Z
N° TVA FR92398197863 RCS BOURGES B398 197 863

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IMMOBILISATION VEST

INDICATIONS:

- Post-traumatic or post-surgical restraint of the elbow and forearm.
- Support and restraint of the arm, with tucked in elbow
- Immobilisation of the upper limb without absolute shoulder immobilisation
- Shoulder subluxation in hemiplegic patients
- Support for a plaster cast.

FEATURES:

Comfortable arm support provided by the padded shoulder strap.
Padded back strap for greater comfort and to prevent shearing.
Height of strap set by buckle adjustment for an optimal fit.
Support tab for the thumb.
Model adaptable to either side.

CONTRAINDICATIONS:

- Must not come into direct contact with broken skin

FITTING THE DEVICE:

- 1 Arrange the forearm inside the sleeve.
 - 2 If desired, place the thumb in the support tab.
 - 2 Adjust and tighten the shoulder strap for optimal restraint.
 - 3 Adjust and tighten the girdle strap for optimal restraint.
- The patient should wear this device directly on a garment.

PRECAUTIONS:

Follow the advice of the health professional who prescribed or supplied the product. If there is doubt regarding use, or discomfort, consult your health professional.
The orthotic device is for personal use.
Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

MAINTENANCE:

Hand wash in 40° water with mild, neutral soap.
Rinse thoroughly without spinning and lay flat to dry, away from excessive heat.
Do not iron.

PRESENTATION:

Blue
5 sizes: (Length from elbow to end of hand)
- Size 0 : 21 – 29 cm - Size 3 : 41 – 49 cm
- Size 1 : 28 – 36 cm - Size 4 : 48 – 56 cm
- Size 2 : 35 – 42 cm

MATERIALS:

Polyester 77% - Polyurethane 12% - Cotton 6% - Polyamide 5%

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