SL 500

INDICATIES:

- Matige tot ernstige lendenpathologieën
- Acute of chronische lendenpijn
- Lumbago
- Ischias

KENMERKEN:

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SL 500

INDICATIONS:

- Moderate to severe lumbar conditions
- Acute or chronic lower back pain
- Lumbago
- Sciatica

FEATURES:

Provides dynamic lumbar support by means of the padded backing and integrated semi-rigid stays for promoting Gate Control. Optimal comfort thanks to its ergonomic hand slits, indentations above the thighs, and deep slope at the back. Additional strap for selective compression according to pain location.

Microfibre material to eliminate sweat by capillary action.

CONTRAINDICATIONS:

- Marked obesity or thinness
- Major respiratory failure
- Prolapse (downward organ displacement)
- Hiatal or inguinal hernia
- Must not come into direct contact with broken skin

FITTING THE DEVICE:

- 1 Open the belt.
 - 2 Put the belt in position, with the label on the inside and at the top.
- The back reinforcements must be positioned one-third above the small of the back, and two-thirds below it.
- 3 Secure the belt so as to achieve the desired support.
 - 4 Place the corrective band over the pain-affected area (D12, L3, S1) and tighten sufficiently to relieve the pain.

The patient should wear this device directly on a garment.

PRECAUTIONS:

Follow the advice of the health professional who prescribed or supplied the product. If there is doubt regarding use, or discomfort, consult your health professional.

The orthotic device is for personal use.

Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

MAINTENANCE:

Hand wash in 40° water with mild, neutral soap.

Rinse thoroughly without spinning, and lay flat to dry, away from excessive heat.

Do not iron.

PRESENTATION:

Grey: 21 cm H or 26 cm H

The 21 or 26 cm height is determined by the height of the patient.

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