SL 700

INDICATIES:

- Matige tot ernstige lendenpathologieën
- Lumbalgie
- Lumbago
- Ischias

KENMERKEN:

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SL 700

INDICATIONS:

- Moderate to severe lumbar conditions
- Lower back pain
- Lumbago
- Sciatica

FEATURES:

Lumbar support belt combining high compression and a padded backing to promote Gate Control.

Optimal comfort thanks to its anatomically cut shape.

Easy to fit thanks to the ergonomic hand slits.

Corrective band for selective compression according to pain location.

CONTRAINDICATIONS:

- Marked obesity or thinness
- Major respiratory failure
- Prolapse (downward organ displacement)
- Hiatal and inguinal hernia
- Must not come into direct contact with broken skin

FITTING THE DEVICE:

- 1 Open the belt.
- 2 Put the belt in position, with the label on the inside and at the top.
The back reinforcements must be positioned one-third above the small of the back, and two-thirds below it.
- 3 Secure the belt so as to achieve the desired support.
- 4 Place the corrective band over the pain-affected area (D12, L3, S1) and tighten sufficiently to relieve the pain.
The patient should wear this device directly on a garment.

PRECAUTIONS:

Follow the advice of the health professional who prescribed or supplied the product. If there is doubt regarding use, or discomfort, consult your health professional.

The orthotic device is for personal use.

Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

To throw away the product, respect strictly the legal regulations of your region.

MAINTENANCE:

Hand wash in 40° water with mild, neutral soap.

Rinse thoroughly without spinning, and lay flat to dry, away from excessive heat.

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