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SL THERMO

INDICATIONS:

- Acute or chronic lower back pain
- Lumbago
- Sciatica
- Moderate to severe lumbar conditions

FEATURES:

Lumbar support belt giving a corset effect thanks to the semi-rigid padded backing.
Semi-rigid padded stays for increasing Gate Control.
User-friendliness thanks to the ergonomic hand slits.

Lumbar belt anatomically cut to reduce friction between the vertebral ribs and the groin when in a seated position.

Supportive lumbar belt that targets pain thanks to an additional detachable strap for selective compression according to pain location.

Heat-regulating, moisture-repellent microfibre material for optimal comfort.

Supportive lumbar belt with combined heat therapy and cold therapy action for rapid pain relief action. This heat therapy/cold therapy action is produced by two hot/cold gel packs:

- Indication for heat therapy: promotes muscle relaxation, rapid pain relief action, promotes vasodilation.

- Indication for cold therapy: promotes the reduction of post-traumatic oedema as a consequence of trauma (muscle tears, traumatism, contusions), promotes vasoconstriction.

CONTRAINDICATIONS:

- Marked obesity or thinness
- Major respiratory failure
- Prolapse (downward organ displacement)
- Hiatal hernia
- Inguinal hernia
- Must not come into direct contact with broken skin

FITTING THE DEVICE:

1 Open the belt.

2 To obtain the heat therapy/cold therapy effect, insert the two gel packs, heated or chilled beforehand, into the sleeves located on the inside of the belt backing.

3 Put the belt in position, with the label on the inside and at the top.

4 Secure the belt so as to achieve the desired support.

5 Place the corrective band over the pain-affected area (D12, L3, S1) and tighten sufficiently to relieve the pain.

The patient should wear this device directly on a garment.

For the heat therapy effect

In the microwave:

Lay the two gel packs flat in the microwave.

Heat the two gel packs for a maximum of 25 seconds at a maximum of 1050 W. Remove the two gel packs from the microwave and manually knead them to distribute the temperature throughout.

If they have not been heated to the desired temperature, repeat the operation for 15 seconds, several times if necessary.

If the two gel packs are too hot, allow to cool.

CAUTION

Never exceed the maximum heating time through repetition.

If the two gel packs begin to swell, switch off the microwave and allow packs gel to cool before removing.

Do not put the belt in the microwave.

For the cold therapy effect

Place the two gel packs in the freezer at -18°C for a maximum of 1 hour.

If the time limit is exceeded, the two gel packs may freeze. To make them soft again, let sit for several minutes at room temperature.

You can also store the two gel packs permanently in the fridge for immediate use.

Do not store the two gel packs in the freezer.

PRECAUTIONS:

Keep out of reach of children. Do not perforate. Not to be taken internally.

Do not use if the two gel packs are damaged.

Never apply the two gel packs directly on the skin.

If you have diabetes or vascular and circulatory problems, you must consult a doctor prior to using the two gel packs.

Follow the advice of the health professional who prescribed or supplied the product. If there is doubt regarding use, or discomfort, consult your health professional.

The orthotic device is for personal use.

Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

MAINTENANCE:

Hand wash in 40° water with mild, neutral soap.

Rinse thoroughly without spinning, and lay flat to dry, away from excessive heat.

Do not iron.

Remove the two gel packs before washing.

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