SL LADY

INDICATIES:

- Matige lendenpathologieën
- Acute of subacute lumbalgie
- Lumbago
- Ischias

KENMERKEN:

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FEATURES:
Provides dynamic lumbar support by means of the padded backing and integrated semi-rigid stays for promoting Gate Control.
Specifically cut to the contours of the female form.
Optimal comfort thanks to its ergonomic hand slits
Vented, breathable fabric
CONTRAINDICATIONS:
- Marked obesity or thinness
- Major respiratory failure
- Prolapse (downward organ displacement)
- Hiatal or inguinal hernia
- Must not come into direct contact with broken skin

FITTING THE DEVICE:
1 Open the belt.
2 Put the belt in position, with the label on the inside.
The back reinforcements must be positioned one-third above the small of the back, and two-thirds below it.
3 Secure the belt so as to achieve the desired support.
4 Place the corrective band over the pain-affected area (D12, L3, S1) and tighten sufficiently to relieve the pain.
The patient should wear this device directly on a garment.

PRECAUTIONS:
Follow the advice of the health professional who prescribed or supplied the product.
If there is doubt regarding use, or discomfort, consult your health professional.
The orthotic device is for personal use.
Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.
To throw away the product, respect strictly the legal regulations of your region.

MAINTENANCE:
Hand wash in 40° water with mild, neutral soap.
Rinse thoroughly without spinning, and lay flat to dry, away from excessive heat.
Do not iron.

PRESENTATION:
Black: 26 cm H
Grey: 26 cm H
5 sizes: (Waist circumference)
- Size 0: 60 - 75 cm
- Size 1: 75 - 90 cm
- Size 2: 91 - 105 cm
- Size 3: 106 - 120 cm
- Size 4: 120 - 135 cm

MATERIALS:
Polyamide 70% - Elastane 16% - Polyurethane 6% - Polyester 4% - Polypropylene 3% - Cotton 1%

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