SL MATERNITY

INDICATIES:

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SL MATERNITY

INDICATIONS:

- Lower back pain related to pregnancy
- Pelvic ligament pain related to pregnancy

FEATURES:

Belt for lumbar and abdominal support. Tightening adjustable according to the month of pregnancy. Adjustable elastic protective band providing lumbar support according to the location of the pain. Belt comprised of four flexible stays. Specifically contoured for pregnant women.

CONTRAINDICATIONS:

- Hiatal hernia
- Lower back pain unrelated to pregnancy
- Incisional hernia
- Must not come into direct contact with broken skin

FITTING THE DEVICE:

- 1 Open the belt.
- 2 Put the belt in position, with the label on the inside and at the top.
- 3 Secure the belt to achieve a moderate degree of abdominal support.
- 4 Place the corrective band over the pain-affected area (D12, L3, S1) and tighten sufficiently to relieve the pain. The patient should wear this device directly on a garment.

PRECAUTIONS:

Follow the advice of the health professional who prescribed or supplied the product. If there is doubt regarding use, or discomfort, consult your health professional.

The orthotic device is for personal use.

Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

To throw away the product, respect strictly the legal regulations of your region.

MAINTENANCE:

Hand wash in 40° water with mild, neutral soap.

Rinse thoroughly without spinning, and lay flat to dry, away from excessive heat.

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