
KLASSIEKE C.H.U.T.

NL

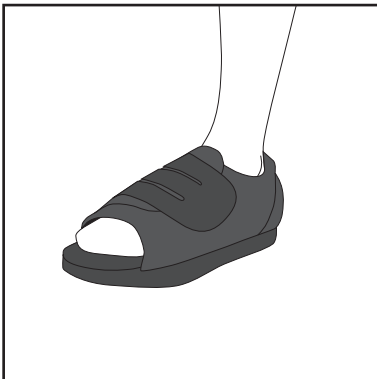INDICATIES:

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SAMENSTELLING:

Polyester 50% - Polyurethaan 30% - Polyamide 20%

CIZETA MEDICALI France SAS
109, rue de la Brasserie - 18200 SAINT AMAND MONTROND
Tel. 02 48 96 87 93 - Fax. 02 48 96 80 91
www.cizetamedicali.fr

Capital social 800 000 €
SIRET 398 197 863 00069 APE 4646Z
N° TVA FR92398197863 RCS BOURGES B398 197 863

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COMPOSITION :

Polyester 50% - Polyuréthane 30% - Polyamide 20%

STANDARD TEMPORARY THERAPEUTIC SHOE

INDICATIONS:

- Oedema
- Lymphoedema
- Following surgery, with foot heavily bandaged
- Prosthetic use
- Significant foot deformity

FEATURES:

Component materials perfectly designed to allow air circulation (facilitates wound healing).

Sole ergonomically shaped for optimal heel support.

Wide-fit shoe, adaptable to foot volume for optimal comfort.

Detachable self-adhesive closure to accommodate numerous foot morphologies and conditions.

Fully opening shoe for easy slip-on.

Rounded, skidproof sole for preventing falls and promoting recovery of natural gait.

Open-toed.

Model fits either foot.

CONTRAINDICATIONS:

- Must not come into direct contact with broken skin

FITTING THE DEVICE:

Pull back the self-adhesive closing strap.

Put on the shoe.

Close and fasten the self-adhesive closing strap.

The patient should wear this device directly on a garment.

PRECAUTIONS:

Follow the advice of the health professional who prescribed or supplied the product. If there is doubt regarding use, or discomfort, consult your health professional.

The orthotic device is for personal use.

Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

To throw away the product, respect strictly the legal regulations of your region.

MAINTENANCE:

Hand wash in 40° water with mild, neutral soap.

Rinse thoroughly without spinning and lay flat to dry, away from excessive heat.

Do not iron.

PRESENTATION:

Black

5 Sizes: (Shoe Size)

- Size 0: 33-35
- Size 1: 35-38
- Size 2: 38-40
- Size 3: 40-43
- Size 4: 43-45

MATERIALS:

Polyester 50% - Polyurethane 30% - Polyamide 20%

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