
GIPSSCHOEN

NL

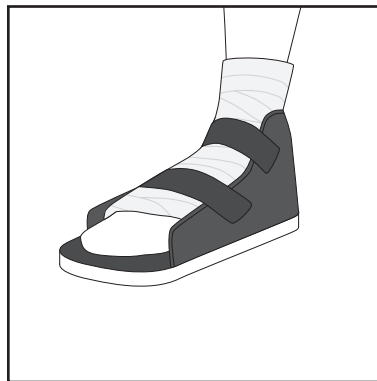INDICATIES:

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SAMENSTELLING:

Polyester 40% - Polyurethaan 30% - Polyamide 20% - Rubber 10%

CIZETA MEDICALI France SAS
109, rue de la Brasserie - 18200 SAINT AMAND MONTROND
Tel. 02 48 96 87 93 - Fax. 02 48 96 80 91
www.cizetamedicali.fr

Capital social 84 000 €
SIRET 398 197 863 00069 APE 4646Z
N° TVA FR92398197863 RCS BOURGES B398 197 863

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Polyester 40% - Polyuréthane 30% - Polyamide 20% - Caoutchouc 10%

PLASTER CAST SHOE

INDICATIONS:
- Following surgery, while wearing a cast
- Cast protection in moving around

FEATURES:
Skidproof sole.
Cast protection in moving around
Self-adhesive closing straps.
Model fits either foot.

CONTRAINDICATIONS:
- Must not come into direct contact with broken skin

FITTING THE DEVICE:
Open the self-adhesive closing straps.
Put your foot into the shoe.
Close and fasten the self-adhesive closing straps.
The patient should wear this device directly on a garment.

PRECAUTIONS:
Follow the advice of the health professional who prescribed or supplied the product. If there is doubt regarding use, or discomfort, consult your health professional.
The orthotic device is for personal use.
Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

MAINTENANCE:
Hand wash in 40° water with mild, neutral soap.
Rinse thoroughly without spinning and lay flat to dry, away from excessive heat.
Do not iron.

PRESENTATION:
Black
4 Sizes: (Shoe Size)
-Size 1: 33-35
-Size 2: 35-40
-Size 3: 40-44
-Size 4: 44-46

MATERIALS:
Polyester 40% - Polyurethane 30% - Polyamide 20% - Rubber 10%

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